


Tips for positioning

1. Apply high on leg.
2. Pant leg should be pulled down taut before applying.
3. Ensure straps are not too tight.

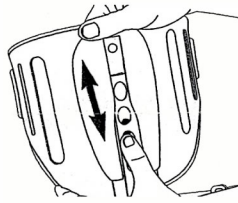


Determine brace length required for the patient (18", 20", 22", or 24")

**STEP
1**

If the required size is 20", skip to Step 2


Adjust the femoral cuff by pressing the blue locking button as shown. Repeat adjustment for tibial cuff.

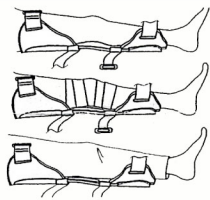

Apply the brace to leg.

**STEP
2**


The brace may be applied over a bare leg (under clothing), over compression dressing, or over clothing.

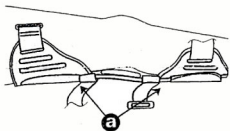

The knee should be centered between the cuffs (as shown to the left)

**STEP
3**


See tips for positioning above

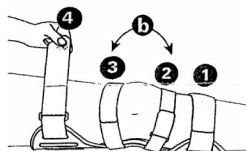

Comfortably secure all four straps in order from bottom to top.


**STEP
4**

Adjust the two middle straps to pull the knee into full extension.

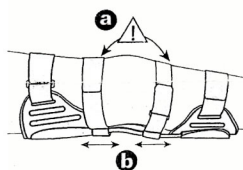

Ensure that middle adjustable straps are not directly over patella area (as shown to the left). Straps should be positioned at least 2 inches away from patella.

**STEP
5**

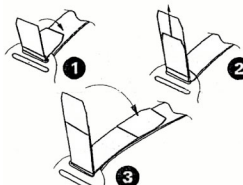

The cuff straps are length adjustable and are preset to the longest length.

**STEP
6**

If a cuff strap is too long, unfasten the hook strip that attaches the strap to the cuffs, move the hook strip farther away from the cuff until the desired length is achieved. Re-fasten the hook strip to the strap.