

Fitting Instructions:



1. Undo toe and calf straps.



2. Step foot into brace.



3. Secure toe strap by affixing the contact closures.



4. Apply calf strap by wrapping behind the leg and affix contact closures.

Sizing Chart:

Size	Women Shoe	Men Shoe
S/M	6 - 10	5 - 9
L/XL	10.5 - 15	9.5 - 14

CAUTION

1. Inspect foot, ankle, and product before and after each use.
2. Do not walk or attempt to place weight on the foot while using this splint without approval from your physician. This is not a weight-bearing product.
3. Do not use outdoors, or on slippery or uneven surfaces.
4. Contact your certified medical professional if any problems occur with the use of this product.
5. Do not use this product if plantar flexion contracture is greater than 10°.
6. This product is not designed to treat uncontrollable, severe plantar flexion spasticity.
7. Patients with foot neuropathy (insensitive feet) should not use this product.

WASHING INSTRUCTIONS

This product may be cleaned using a mild detergent and cold water. Do not wash in a washing machine. Air dry. Do not place in a dryer or near heater.