

ULTRALITE SPINE SYSTEM

Code L0637 Approved

ITEM # **99524**
SIZES: SMALL -4/5X

Fitting & User Instructions

Orthotist Instructions

1) Attach the brace to the UltraLite Spine System frame by sliding panel ends between attachment straps and center the brace.

2) Fit the system on the patient,

a- Position the Anterior Panel

b- Pull tab to create compression on patient

c- Mark the anterior panel position relative to the inside left panel

Remove the orthosis from the patient.

3) Attach the anterior panel to its marked position on the interior of the left brace panel by snugly wrapping the velcro strap first from the bottom-up, then from the top-down.

User Instructions

1) Fasten the adjust handle close to the pulley unit. The handle is always on the left.

2) Stand and hold the brace by its side panels, parallel to the floor.

3) Wrap the brace around the body; determine the best and most supportive vertical position.

4) Pull slightly outward on side panels to maximize open position before using fabric fastener to close the front; right over left.


5) Pull handle away from fabric and across the body, place it anywhere on fabric fastener strip to achieve desired support.

6) Attaching the Bell-Horn UltraLite Spine System Chairback Frame insert:

a- Place LSO around the outside circumference of Chairback Frame insert, keeping the handle on the left side.

b- Align LSO making sure the Pulley System Unit is centered on Chairback Frame Insert.

c- Fasten posterior support straps to secure the LSO to the Chairback Frame insert.

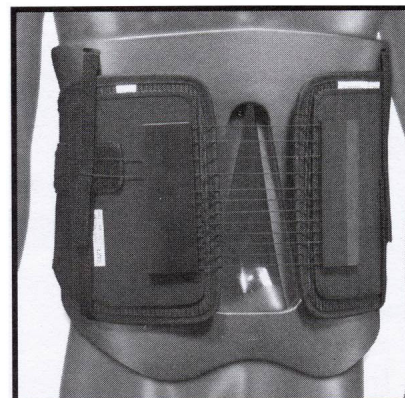

For Maximum Effectiveness

- Before putting on the UltraLite Spine System make sure the side panels are extended or pulled as far apart as possible.
- The handle should always be on the left.
- Always start with the handle near the pulley system.
- The UltraLite Spine System frame must be centered on the spine.
- Experiment with adjustments by releasing or pulling the handle as you change position.

Cleaning

- Hand wash in warm water, towel off and air dry

